

CaviCide1™

If gross debris is present on the surface:

1. Spray CaviCide1 directly onto surface to be cleaned.



2. Wipe with clean towel to pre-clean surface of all gross debris. Once surface is clean, dispose of towel in accordance with Federal, State, and local regulations for infectious materials disposal.



Disinfect after Cleaning

1. Spray CaviCide1 directly onto surface, thoroughly wetting area. Leave surface wet for 1 minute.




2. Wipe with clean towel. Once complete, dispose of used towel.




If NO gross debris is present on the surface:

1. Spray CaviCide1 directly onto surface, thoroughly wetting area. Leave surface wet for 1 minute.

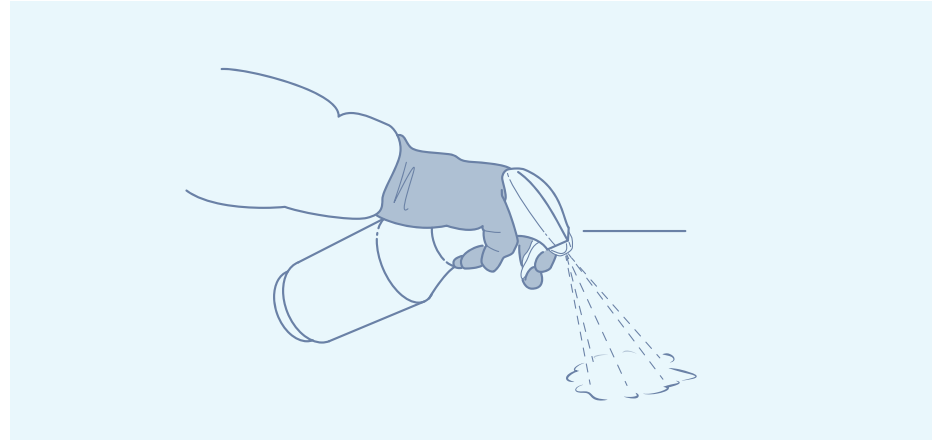

2. Wipe with clean towel. Once complete, dispose of used towel in accordance with Federal, State, and local regulations for infectious materials disposal.

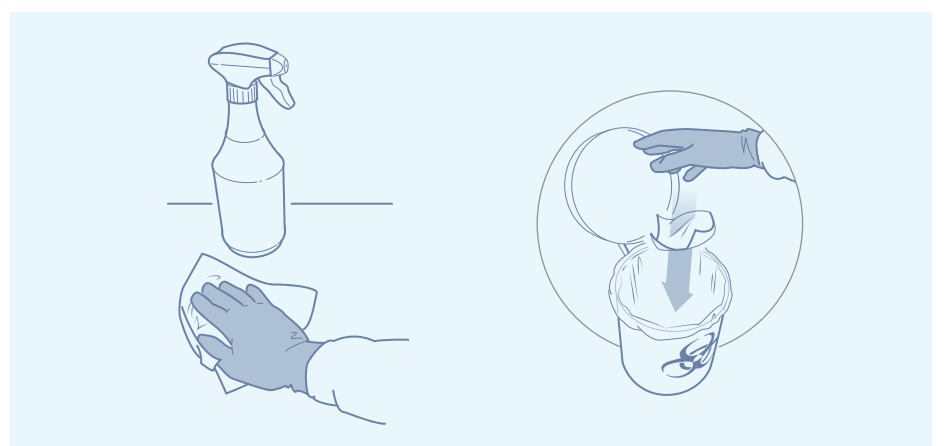

Ordering Information

Part No.	Description	Packaging
13-5002	CaviCide1 – 2 ounce spray	48 bottles / case
13-5024	CaviCide1 – 24 ounce spray	12 bottles / case
13-5000	CaviCide1 – 1 gallon	4 bottles / case
13-5025	CaviCide1 – 2.5 gallon	2 bottles / case
13-5055	CaviCide1 – 55 gallon	1 drum

