



# MultiZym


**With triple enzyme power!**

Detergent, liquid concentrate

Medical



**Non  
Hazardous**

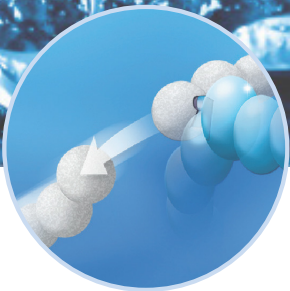

**FREE OF BORIC ACID AND BORATES**

5 L



5 l canister  
4087 35

**Excellent cleaning performance due to multi-enzymatic formula.**

Active residue-free cleaning, prevents redeposition of protein residues.  
Removes biofilms.

Excellent material compatibility, pH-neutral.

**neodisher® MultiZym** is not a hazardous product according to the CLP-Regulation (EC) No 1272/2008.

# neodisher® MultiZym – active enzymes for manual instrument reprocessing.


Multi-enzymatic detergent with protease, lipase and amylase

## Our formula for success:

- The formula of **neodisher® MultiZym** combines three classes of enzyme, thus guaranteeing maximum cleaning power with a synergistic effect for all types of residue: protease breaks down proteins from protein residues, lipase removes fat-containing contaminants and amylase removes polysaccharides, for example from bronchial mucus.
- For manual cleaning of thermostable and thermolabile instruments, including laparoscopic and micro-instruments, dental instruments, flexible endoscopes, anaesthetic equipment, containers and other medical utensils in immersion baths and ultrasonic baths

## Application recommendation:

Manual cleaning with normal levels of contamination	1 - 2.5 ml/l, 15 - 50 °C, 2 - 10 min
Manual cleaning with high levels of contamination	10 - 30 ml/l, 15 - 50 °C, 2 - 10 min
Manual cleaning in ultrasonic baths	1 - 30 ml/l, 15 - 50 °C, 2 - 10 min

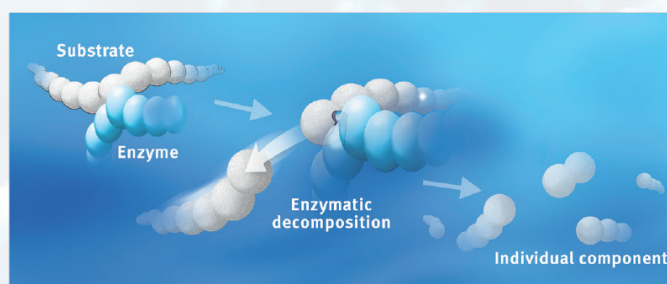

Enzymes work in accordance with the 'lock and key' principle: each enzyme cleaves one specific substrate. Like a key, the enzyme's structure 'fits' one particular type of substrate.

## Our Competence – Your Benefit

- first-class, residue-free cleaning performance
- multi-enzymatic formula with protease, lipase and amylase
- **free of boric acid and borates, perfume-free with no colourings**; instruments are clearly visible in the solution

- practical one-litre bottle with integrated dosing aid

- independently assures excellent decontamination with regard to organic materials, prevents redeposition of protein residues
- effectiveness of active ingredients, high efficiency and exceptional cleaning results even at low concentrations
- very user-friendly with exceptional material compatibility: suitable for surgical stainless steel, optics, standard synthetic materials, anodised aluminium, silicone and materials used in anaesthetic equipment
- easy to use and simple dosing without additional accessories

US1166 / 02-2019 © Dr. Weigert, Hamburg

Product code	Type	Size	Price 1-4	5-9	10+
C-NMZ	MultiZym Detergent	5 Liter	\$79.00	\$75.05	\$71.11
Add Neodisher PreStop Instrument Protector & PreCleaning Solution, 750ml spray bottle					



Purchase online