


T-Edge Quick Guide

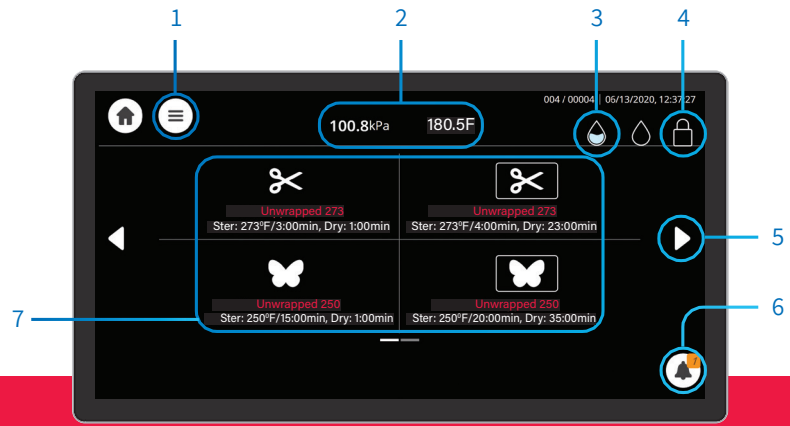

Main Screen Orientation

- | | | |
|----------------------------|--------------------------------|--------------------------------|
| 1 Settings | 7 Changing Cycles (Class S) | 7 Changing Cycles (Class B) |
| 2 Pressure / Temp. Gauge | ✂ Unwrapped Instruments 270° F | ✂ Unwrapped Instruments 273° F |
| 3 Water Level | ✂ Wrapped Pouches 270° F | ✂ Wrapped Pouches 273° F |
| 4 Door Status | 🦋 Handpieces | 🦋 Wrapped Delicate 250° F |
| 5 Toggle Between Screens | 🦋 Unwrapped delicate 250° F | 🦋 Unwrapped Delicate 250° F |
| 6 Alerts | | |

Choosing the sterilization cycle



1 Insert the pouches or instruments you want to sterilize



2 Choose the required cycle in the main screen




4 Press "Start" to begin the sterilization cycle



3 Close the door



5 The cycle status will be displayed on the "Status Bar". You can stop the cycle at any point by pressing the "Stop" button

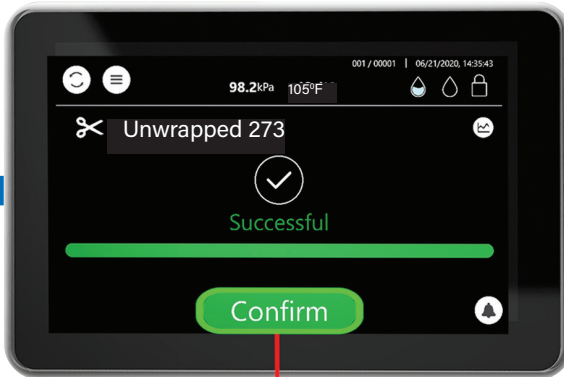

6 Once the sterilization cycle is complete - press the "Confirm" button in order to open the door.



8 In order to efficiently clean the autoclave chamber (once a week or every 25 cycles), choose "Chamber Clean" cycle and place a Chamber Clean tablet at the back of the chamber. Close the door and begin the cycle. After cycle has ended, you must run a second cycle of just water.



7 If alerts show on the screen - press the alert (bell) icon. The alert will be listed on the screen.